# TRANSONIC GEL

## ES. GEL PARA ULTRASONIDOS

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# TRANSONIC GEL

## EN. ULTRASOUND GEL

Gel for ultrasound diagnosis and therapy.

REF		ml	Packaging	Dispenser	
Blue	G-15	GC-15	250	Bottle	
	G-15/05	GC-15/05	500	Bottle	
	G-15/1	GC-15/1	1000	Bottle	
	G-15/5	GC-15/5	5000	Flexible container	
	G-15/5RB	GC/15/5RB	5000	Rigid container	

### INSTRUCTIONS FOR USE

1. Clean and dry the area of application, if necessary.
2. Apply to the selected areas, spreading the gel evenly.
3. Remove with a disposable towel and dispose of the product in accordance with the corresponding protocol.

In the case of 5L format, before use, replace the cap of the container with the dosing pump or nozzle and refill the 250 ml bottle provided with the product.

### TIME FOR USE

The maximum time for use is 60 minutes.

### WARNINGS

This product must be used by authorised medical staff or by personnel accredited for the use of ultrasound gel. – Follow the instructions for use and disposal of the product. – If possible, do not apply the ultrasound gel to damaged skin. – Misuse or improper application of the ultrasound gel can cause an incident or ineffective therapy. – The temperature limits recommended for its use are between 5°C (41°F) and 40°C (104°F). – Non-sterile product. Non-sterilisable product. – Check the expiry date on the container.

### DISPOSAL

- Dispose of the product following the hospital protocol for contaminated waste, in accordance with the current standards.
- Dispose of the packaging following the guidelines for the management of recyclable waste.

### WARRANTY

TELIC, S.A.U. guarantees that the product complies with the Regulation (EU) 2017/745 on Medical devices and was manufactured following the procedures of the TELIC, S.A.U. quality management system certified under ISO standard 13485. TELIC, S.A.U. accepts no liability for any medical costs or direct or indirect harm due to the lack of operation or incorrect operation of the products when used without observing the instructions for use. We advise you to duly report any operational problem or product fault to the Quality Assurance Department of TELIC, S.A.U. (incidents@telic.es). In the event of any serious incident related to the product, this must be reported immediately to TELIC, S.A.U. and/or the competent authority of the Member State in which the incident occurs.

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