

Firming cream for radiofrequency

OXD

> Product description

OXD firming cream with formulation and creamy texture compatible with radiofrequency treatments. Pleasant non-greasy formula with squalane, a powerful antioxidant and hydrating substance that provides easy absorption without leaving a greasy residue and that nourishes, softens and seals the skin's moisture to protect it.



> Product characteristics

- Pleasant non-greasy formulation, suitable for either body or facial treatments.
- Contains squalane powerful antioxidant and hydrating substance, well known for its regenerating and firming properties. Enhances performance of radiofrequency treatment.
- Creamy-gel texture makes radiofrequency treatments easier. High water content of the formulation refreshes skin and reduces warming during the treatment. Suitable for monopolar and/or bipolar radiofrequency equipment.
- High conductivity with low salt levels.

> Technical specifications

> Composition (INCI)

Aqua (Water), Glycerin, Squalane, Polyglyceryl-6 Distearate, Coco-Caprylate/Caprates, Betaine, Phenethyl Alcohol, Jojoba Esters, Acrylates/C10-30 Alkyl Acrylate Crosspolymer, Cetyl Alcohol, Caprylyl Glycol, Parfum, Cocamidopropyl Betaine, Polyglyceryl-3 Beeswax, Sodium Hydroxide, Trisodium Ethylenediamine Disuccinate, Sodium Benzoate.

> Physico-chemical properties

- Colour, Odour, Appearance: Viscous creamy-gel, bright white colour. Characteristic odour to its fragrance.
- Viscosity: 100.000 – 200.000 cps.
- pH: 5,5 – 6,0.
- Density: 0,98 – 1,00 g/mL.
- Conductivity: > 1.000 μ S/cm.

> Microbiological properties

- Total aerobes: <100 CFU/mL.
- Moulds and yeasts: <100 CFU/mL.
- Pathogens (Staphylococcus aureus, Candida albicans, Pseudomonas aeruginosa, Escherichia coli): Absence.

> PAO and expiry date

- Period after opening (PAO): 12 months after aperture.
- Expiration: 3 years upon the manufacturing date.

> Packaging

#	Packaging
TFCR1	White HDPE bottle of 1000 mL with dosing pump.

> Instructions for use**> Indications**

- This product can be applied in the entire body, before radiofrequency treatments.
- Product intended for all skin types.

> Warnings

- For external use only.
- Do not apply on injured skin, eyelids or mucous membranes.
- Do don't swallow.
- Keep out of the reach of children.

Date: May 20223

Specification N°: ETP/213/9

> Regulatory information

TELIC, S.A.U. guarantees that this product is in conformity with the **European Regulation 1223/2009 for Cosmetic Products** and that it has been manufactured following the directives of the Quality Assurance System based on **ISO 22716**.

CPNP Code: 2378872