



Product

DORMO COLD/HOT

Reusable pack for cold/hot

Date

July 2021

References

TTG00: small pouch, FC-02

TTG01: big pouch, FC-01



Specification N.

ETP/221/06

TECHNICAL SPECIFICATIONS

Materials

- Pouch: OPA/PE plastic film
- Contents: blue viscose gel, water based

Main dimensional properties

- FC-01: 130mm x 260mm
- FC-02: 130mm x 180mm

Packaging

- FC-01: service box of 20 units
- FC-02: service box of 20 units



PRODUCT CHARACTERISTICS

Properties

- Cold and hot use
- Doesn't become rigid when frozen
- Maintains viscosity when hot
- May be kept in freezer until next use
- Do not apply directly on skin: wrap in a dry towel or in a proper cover prior to use.

Cold use

Treatment with cooling effect:

- The cooling effect of the product helps reduce inflammation and pain. It is used to treat:
 - Swelling
 - Acute injuries (haematomas and contusions)
 - Muscle or joint injuries
 - Fever (it is not designed to lower the body temperature)

Preparation:

- The steps to use the reusable cooling pack are described below:
- Place the pack in the freezer for 1 or 2 hours.
- Remove the pack from the freezer and wrap it in the reusable cover or a dry towel.
- Apply to the affected area.
- The pack can be returned to the freezer for storage.

Hot use

Treatment with heating effect:

- The heating effect of the product helps reduce muscle and joint pain. It is also suitable for lower back pain, torticollis, rheumatism and hypothermia. It is used to treat:



- Neck or shoulder pain
- Chills (used as a heater)
- To relieve muscle tension due to stomachache Lower back pains, torticollis, rheumatism, hypothermia

Preparation:**Bain-marie:**

- Bring water to the boil in a pan.
- Remove from the heat.
- Submerge the pack in the hot water for 5 to 10 minutes.
- Remove the pack from the water and wrap it in the reusable cover or a dry towel.
- Apply to the affected area.

Microwave:

- Submerge the bag in water completely in a microwave-appropriate container.
- Heat at intervals of 30 seconds at maximum power (800-900 W) until the desired temperature is reached.
- Remove the pack from the water and wrap it in the reusable cover or a dry towel.
- Apply to the affected area.

REGULATORY INFORMATION

TELIC, S.A.U. guarantees that this product is in conformity with **Directive 93/42/EEC** and that it has been manufactured following the directives of the Quality Assurance System certified as **ISO 13485**.

This product is classified as:

- **Class I, non-sterile** product according to Anexo VIII del reglamento (UE) 2017/745, Rule 1.
- GMDN code: 37240 Hot/cold therapy pack, reusable.
- ENDN code: V9099 (Various Devices not included in other classes-other).