## EN. BIOPSY PUNCH STERILE

GMDN Code: 35286

Sizes: 1mm, 2mm, 3mm, 4mm, 5mm, 6mm, 7mm, 8mm.

Stainless steel, ABS plastic/ LDPE

Sterilized by gamma radiation of minimum 25 kGy and not more than 32 kGy (2.5 M Rads to 3.2 M Rads)

### INSTRUCTIONS FOR USE

#### INTENDED USE

The device is used for obtaining full-thickness skin specimen for the biopsy test.

The device is classified as surgically invasive and for transient use.

#### MODE OF ACTION

Biopsy punch is rotated down through the epidermis and dermis, and into the subcutaneous fat, to take the sample from tissue for the examination. Sharp angle cut tube facilitates the extration. It can be use directly after opening the pouch.

#### INDICATIONS

- For obtaining diagnostic full-thickness skin specimen.
- Different size for different skin area of body.
- Removal of skin specimen from operating area.

#### CONTRAINDICATIONS

Reused device can transmit contagious diseases to patient and/or user.

Adverse event may happen if device is used after expiry date, as expiry date means the latest day of guaranteed sterility.

Do not use in cardiac / ophthalmic surgery.

#### INSTRUCTIONS

- Select proper type and size of biopsy punch as per skin area and size of specimen.
- Inspect the package to check that the shelf life is valid and remove a pouch from the package.
- Peel off the pouch and remove the device.
- Clean the selected area of the skin with a suitable disinfectant.
- Punching must be done by rotating punch on the skin with a gentle pressure.
- Remove the biopsy punch after cutting the desired depth of the skin.
- Take the sample of specimen.

#### WARNING

Before use, always check the integrity of the product and packing, along with expiry date. For single use only. Reused device can transmit contagious diseases to patient and/or user.

Once the pouch is opened, use the device immediately.

The manufacturer is not responsible for any possible consequences because of an improper use of the device. Neither holds responsibility if the device is resterilized or reused.

Sterility of the device is not guaranteed in case that the pouch is broken or torn. Resterilization or reuse provoke changes in the used material and mechanical properties. The intended use can also be altered, as blade sharpness may be blunt.

Keep out of the reach of children.

Used devices should be handled according to the country bio-waste rules and laws.

#### PRECAUTIONS

Always open the pouch tearing from the indicated symbol.

Devices are extremely sharp, use care while handling. Care must be taken so that the pouch is not opened in an unsterile area otherwise the device which has already been sterilized by gamma radiation will become unsterile. Proper procedures must be used as applicable for handling any sterile product.

To prevent any injury or accident, handle with care during the use of the device. Also do so during the disposal, to avoid any contact or injury due to its sharp nature.

In case that performance of the device does not fulfil the intended use, replace it with new one.

#### KNOWN CHARACTERISTICS OF DEVICE IN CASE OF REUSE

Difficulty to scrap lesions from the surface of the patient's skin.

Risk to transmit any infectious disease.

#### STORAGE CONDITION

- Keep away from direct sun light.
- Keep away from rain.
- Storage temperature should be 10 to 40°C.

- Humidity of storage area should be 35% RH to 65 % RH.
- Keep away from children.
- Store in cool and dry place.

#### DISPOSAL SYSTEM

Discard the device in proper waste container and dispose off the product in accordance with the accepted medical practice and applicable local, state and country laws and regulations for handling of bio-medical waste.

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TELIC France SAS
EC REP MDSS GmbH, Schiffgraben 41,