DORMO®

ES. GEL LUBRICANTE HIDROSOLUBLE

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DORMO®

EN. HYDROSOLUBLE LUBRICANT GEL

Lubricant gel for catheters and general hospital procedures. It is sold with the following references and formats.

REF	Description
G-20	Hydrosoluble lubricant gel. 100 ml tube.
G-20/5RB	Hydrosoluble lubricant gel. Rigid 5000 ml container with dosing pump.

Recommended for use with catheters, enemas and vaginal or anal douches, gynaecological evaluations or instrumental or digital examinations. For general use with all patient types.

Transparent hydrosoluble gel. Easy to spread and excellent adhesion. Non-irritant. Does not damage rubber, metal devices, instruments or natural textile or synthetic materials.

INSTRUCTIONS FOR USE

1. Apply to the selected area or instruments, spreading the gel evenly. Perform the medical procedure in accordance with the usual protocols.
2. When appropriate, remove the remaining product from the instruments with a disposable wipe and clean according to the instructions provided by the device manufacturer.

TIME FOR USE

The maximum time for use is 60 minutes.

WARNINGS

This product must be used by authorised medical staff or by personnel qualified to carry out the aforementioned medical procedures. – This gel does not have contraceptive properties. - Follow the instructions for use and disposal of the product after use. - Misuse or improper application of the lubricant gel may cause incidents and/or ineffective treatment or therapy. – In case of allergy or any other adverse reaction, consult a doctor immediately. - Do not submerge, rinse or sterilise this product, as they are procedures that can leave toxic residue or cause poor functioning. Neither the product design nor the materials with which it is manufactured are compatible with conventional cleaning and sterilisation procedures. – Do not use the product if the gel is dry or has changed colour. –In cases of accidental ingestion, consult a doctor immediately. - The recommended temperature limits for storage are between 5°C (41°F) and 40°C (104°F). – Non-sterile product. Non-sterilisable product. – Check the expiry date on the container before use.

DISPOSAL

- Dispose of the product following the hospital protocol for contaminated waste, in accordance with the current standards.
- Dispose of the packaging following recyclable waste management.

WARRANTY

TELIC, S.A.U. guarantees that the product complies with the Regulation (EU) 2017/745 on Medical devices and was manufactured following the procedures of the TELIC, S.A.U. quality management system certified under ISO standard 13485. TELIC, S.A.U. accepts no liability for any medical costs or direct or indirect harm due to the lack of operation or incorrect operation of the products when used without observing the instructions for use. We advise you to duly report any operational problem or product fault to the Quality Assurance Department of TELIC, S.A.U. In the event of any serious incident related to the product, this must be reported immediately to TELIC, S.A.U. (incidents@telic.es) and/or the competent authority of the Member State in which the incident occurs.

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