
ES. ROTULADOR DERMOGRÁFICO
EN. SKIN MARKER
FR. MARQUEUR CUTANÉ
IT. PENNARELLO DERMOGRAFICO
DE. HAUTMARKIERSTIFT
PT. MARCADOR DE PELE

Referencia Reference Référence Riferimenti Produktversionen Referência	Descripción Description Description Produktbeschreibung Descrição	Presentación Presentation Conditionnement Presentazione Präsentation Apresentação	Esterilidad Sterility Stérilité Sterilità Sterilidade
RQ-01 TO	Rotulador dermatográfico. Punta estándar con regla. Skin marker. Standard tip with ruler. Marqueur cutané. Pointe standard avec règle. Pennarello dermatografico. Punta normale con righello. Hautmarkierstift. Standardspitze mit lineal. Marcador de pele. Ponta padrão com régua.	Bolsa individual. Individual pouch. Sachet individuel. Confezione singola. Einzelner beutel. Bolsa individual.	STERILE R
RQ-05 TO	Rotulador dermatográfico. Doble punta con regla. Skin marker. Dual tip with ruler. Marqueur cutané. Double pointe avec règle. Pennarello dermatografico. Punta doppia con righello. Hautmarkierstift. Doppelspitze mit lineal. Marcador de pele. Ponta dupla com régua.	Bolsa individual. Individual pouch. Sachet individuel. Confezione singola. Einzelner beutel. Bolsa individual.	STERILE R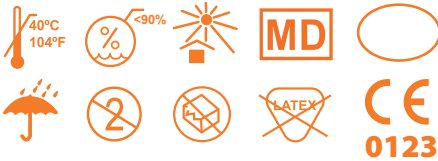

TELIC, SAU
C/ Moli d'en Barri, 7
08415 Bigues i Riells
Barcelona, Spain
www.telic-group

WITH THE GUARANTEE OF
TELIC GROUP

Dongguan Tondaus Meditch Co., Ltd
E-mail: leo@tondaus.com
T: +86 769 22987190 Contact: Leo
Add: 3rd Floor, No.27, Rd. Guantai, Chiling
Community, Houjie Town, Dongguan,
Guangdong, China.
Postal code: 523900

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EN. SKIN MARKER

DESCRIPTION
The skin marker is a medical device used to mark the skin of the patient before or during a surgical procedure.

REFERENCES
For more details of the different models, see the references table.

CHARACTERISTICS
The skin marker is used to mark or select the cutting area before or during the surgical procedure. Long-lasting ink, scratch-resistant and practically odorless. It contains basically gentian violet, which is neither toxic, nor allergenic, nor irritating. Single use product, difficult to erase during the disinfection procedure. Sterile references (RQ-01 TO, RQ-05 TO) are sterilized by radiation. Quick, easy and convenient use. It can be used in general surgery, plastic surgery, orthopedic surgery, neurosurgery, cardiovascular surgery and radiotherapy.

INTENDED USE
Marking and selection of the dry skin on the intervention area for diagnosis and radiological interventions, surgery and treatment of skin diseases.

INSTRUCTIONS FOR USE
1. Clean and dry the patient's skin, (Do not use the skin marker neither the ruler on wet skin).
2. Mark the skin.
3. Use iodine to disinfect the skin and set the marking.
4. Use 75% alcohol to disinfect the skin and perform the procedure.
5. Remove the marking from the skin with alcohol if needed.

WARNINGS
1. Consider the possibility that the patient is allergic to gentian violet. In case the patient experiences discomfort during the use of the product, stop the application and treat properly.
2. Disposable product of single use only to prevent infections. Do not reuse. Do not re-sterilize.
3. The surgical marker and the ruler should be used only on dry skin. In case of wet skin or sweat, dry the skin and proceed according to the instructions for use.
4. Protect tip during use and cover it with the cap when it is not used to prevent the ink from drying.
5. Use the product immediately after opening the sterile package. After use, discard the product according to current hospital protocols. Do not use if the package is dirty or damaged.
6. Do not dry or clean the product with products that contain organic solvents, this may damage the product.

CONTRAINDICATIONS
Pay special attention in case of patients with injuries or damaged mucosa or skin. Do not use the skin marker neither the ruler on wet skin or surface.

POSSIBLE COMPLICATIONS
None.

STORAGE
- Store in a fresh and dry place. Avoid exposure to water, direct sunlight, extreme temperatures or humidity and avoid contact with corrosive gases during storage.
- Recommended temperature: <40°C
- Recommended relative humidity: < = 90%
- Sterility is guaranteed for 3 years. Consult the expiration date on the package.

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