DORMO®-TENS

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EN. PRE-GELLED MUSCLE STIMULATION ELECTRODES

Dormo® -Tens Pre-gelled muscle stimulation electrodes are adhesive electrodes with conductive gel which have been designed for electro-stimulation use in physiotherapy and beauty-care treatments. Electrodes are indicated for use with transcutaneous electrical stimulation devices. Some common types of transcutaneous stimulation devices include, but are not limited to, Transepithelial Nerve Stimulation (TENS) and electrical muscle stimulation (EMS) devices. Transcutaneous neuro-stimulation electrodes are passive devices serving as an interface between a user's skin and a neuro-stimulation device.

INSTRUCTIONS FOR USE:

FIGURE 1 - Verify that the package containing the electrodes or gel replacements is not damaged; in case of damages do not use the product. - Prepare the application area before use: it must always be clean, dry and free from hair. Lotions or oily skin will decrease the electrode adhesion. Avoid the use of solvents as these could cause skin irritation and reduce adherence of electrode. In case of using alcohol, make sure that this has completely dried up before applying the electrodes.

FIGURE 2 - Connect the electrodes to the electro-stimulator using adequate lead wires. Check that the leads and connections are not damaged, metal areas cannot be exposed.

FIGURE 3/4 - Carefully remove the electrode from protective liner by lifting a corner of electrode and slowly peel it away. Avoid touching the gel as much as possible. Do not peel the electrode by wire, pulling the wire might lead to damages in the connections.

- Stick the electrodes on the specified area: center first, then smooth down to electrode edges, press firmly. Electrodes must be placed following the instructions of the physician:

- Turn the stimulator on and adjust according to stimulator and physician's instructions for treatment. Refer to the user manual of the stimulator for further details.

- When the stimulation session is complete, turn the stimulator off and remove the electrodes.

- In case that the procedure is interrupted, check the functioning of the system again before restarting the session. Check that electrodes are conveniently adhered to the skin surface.

TIME FOR USE

In accordance with the medical procedures, the electrodes or the gel replacements of the electrodes must be changed if deterioration is detected during operation or after a maximum of 24h.

ELECTRODE REMOVAL:

- Once treatment is completed, turn off the electrical stimulator.

- Lift at the edge of the electrode and carefully peel to remove electrode from the application area. A quick removal of the electrode could cause skin traumas. Do not remove electrodes by pulling the cable as damages might be produced in the product.

- Disconnect stimulator lead wires from the electrode connectors.

- Clean the electrodes after each use according to maintenance instructions. If electrodes begin to lose adhesion, replace with new electrodes

- Place the electrodes back on the protective liner and keep into the storage bag. Reseal tightly to prevent dry out.

- Disconnect the leads from the stimulation device.

HYDROGEL REPLACEMENTS FOR DT SERIES ELECTRODES:

- Discard the worn hydrogel from the silicone electrodes and clean the surface with a cloth dampened with water or alcohol. - Remove the white silicone protector from the top of the replacement hydrogel. - Place the silicone electrodes on the top of the hydrogel so that the entire surface of the electrode is fully covered and press firmly so that the hydrogel securely adheres to the electrode. - Use the electrodes according to the usual procedure.

- Discard the worn hydrogel from the silicone electrodes and clean the surface with a cloth dampened with water or alcohol. - Remove the white silicone protector from the top of the replacement hydrogel. - Place the silicone electrodes on the top of the hydrogel so that the entire surface of the electrode is fully covered and press firmly so that the hydrogel securely adheres to the electrode. - Use the electrodes according to the usual procedure.

SILICON REPLACEMENTS FOR DT SERIES ELECTRODES:

- Cut the silicon stripe in to the usage size. - Place the hydrogel replacements on the top of the silicon so that the entire surface of the electrode is fully covered, and press firmly so that the hydrogel securely adheres to the electrode. In case that a conductive gel is used, ensure that the whole surface is covered. - Use the electrodes according to the usual procedure.

STORAGE AND MAINTENANCE:

FIGURE 5 - To maintain the adhesion of the electrodes, the surface of the gel should be cleaned after each use: wet the gel with a drop of water, let dry for a few seconds, and return the electrodes to the silicon side of the backing. - If the adhesive hydrogel is over-saturated, let the electrode air out in the refrigerator with the hydrogel facing upward until it recovers its adhesiveness.

FIGURE 6 - Once opened, store the electrodes in a cool, dry place, in their tightly closed original packaging, away from direct sunlight. - Duration of use of the electrodes is up to 20 times approximately. The times of use of the electrode may vary depending on the skin conditions, skin preparation, type of stimulation, storage conditions.

WARNINGS AND PRECAUTIONS:

Only use equipment specially designed for electrostimulation with a low intensity current (< 0.1 Watts/cm²). Read the stimulation device user manual carefully and check the system is working before starting the procedure.

The muscle stimulation electrodes must only be used by a health professional familiar with their proper use and application, or under their direction. - Do not use the product if you see any imperfections in the electrode or connecting cables. - Do not use if the gel is dry or not homogenous, if there are problems with adherence or it presents any other alteration. - The product must not be used on pregnant women, or epileptic patients with pacemakers, internal defibrillators or other implanted metal products. - During the procedure, always select the lowest possible energy level. If the stimulation capacity is lower than normal, do not increase the output power without first checking that the connectors and cable are connected correctly and that the whole electrode surface is in contact with the patient's skin. - Do not place the electrodes close to wounds, recent scars or damaged skin, near the eyes, on bone protuberances or on metal prostheses. Do not place the electrodes in such a way that the electric current would pass through the head, the carotid (at the front of the neck) or the heart. - Keep the electrodes separated during the treatment. - Reduced contact between the skin and the electrode can focus the electric current on one point and cause burns or other adverse reactions.

Do not use electrodes that are not a suitable size or shape for your device. Always apply them correctly. - It is not recommended to use this product in patients with progressive episodes of skin irritation or contact allergies. - Stop the treatment if the patient feels discomfort or a burning sensation, and seek medical attention. Do not use the electrodes during activities that require constant attention (e.g. while operating heavy machinery, driving on a motorway, etc.). - Keep the electrodes out of the reach of children. - Follow the instructions for use and disposal of the product. - For use only on clean, unbroken skin (do not use on open wounds, lesions or infected or inflamed areas). - The product is not suitable for internal use. - In the event of accidental ingestion, consult a doctor immediately. - The electrodes are for use on one person only. They can be repositioned several times. Reuse of the electrodes for different patients can cause cross-contamination. - Do not submerge, rinse or sterilise this product, as they are procedures that can leave toxic residue or cause poor functioning. Neither the product design nor the materials with which it is manufactured are compatible with conventional cleaning and sterilisation procedures. - The temperature limits recommended for storage are: DT-XX / RT-XX / CSC-XX: 5°C (41°F) - 40°C (104°F); SC-XX / ST-XX: 5°C (41°F) - 30°C (86°F). - The recommended relative humidity for storage is between 20% and 80%. - Check the expiry date on the container. - Misuse or incorrect application of the muscle stimulation electrodes can cause undesired effects or ineffective therapy.

Do not use electrodes that are not a suitable size or shape for your device. Always apply them correctly. - It is not recommended to use this product in patients with progressive episodes of skin irritation or contact allergies. - Stop the treatment if the patient feels discomfort or a burning sensation, and seek medical attention. Do not use the electrodes during activities that require constant attention (e.g. while operating heavy machinery, driving on a motorway, etc.). - Keep the electrodes out of the reach of children. - Follow the instructions for use and disposal of the product. - For use only on clean, unbroken skin (do not use on open wounds, lesions or infected or inflamed areas). - The product is not suitable for internal use. - In the event of accidental ingestion, consult a doctor immediately. - The electrodes are for use on one person only. They can be repositioned several times. Reuse of the electrodes for different patients can cause cross-contamination. - Do not submerge, rinse or sterilise this product, as they are procedures that can leave toxic residue or cause poor functioning. Neither the product design nor the materials with which it is manufactured are compatible with conventional cleaning and sterilisation procedures. - The temperature limits recommended for storage are: DT-XX / RT-XX / CSC-XX: 5°C (41°F) - 40°C (104°F); SC-XX / ST-XX: 5°C (41°F) - 30°C (86°F). - The recommended relative humidity for storage is between 20% and 80%. - Check the expiry date on the container. - Misuse or incorrect application of the muscle stimulation electrodes can cause undesired effects or ineffective therapy.

Do not use electrodes that are not a suitable size or shape for your device. Always apply them correctly. - It is not recommended to use this product in patients with progressive episodes of skin irritation or contact allergies. - Stop the treatment if the patient feels discomfort or a burning sensation, and seek medical attention. Do not use the electrodes during activities that require constant attention (e.g. while operating heavy machinery, driving on a motorway, etc.). - Keep the electrodes out of the reach of children. - Follow the instructions for use and disposal of the product. - For use only on clean, unbroken skin (do not use on open wounds, lesions or infected or inflamed areas). - The product is not suitable for internal use. - In the event of accidental ingestion, consult a doctor immediately. - The electrodes are for use on one person only. They can be repositioned several times. Reuse of the electrodes for different patients can cause cross-contamination. - Do not submerge, rinse or sterilise this product, as they are procedures that can leave toxic residue or cause poor functioning. Neither the product design nor the materials with which it is manufactured are compatible with conventional cleaning and sterilisation procedures. - The temperature limits recommended for storage are: DT-XX / RT-XX / CSC-XX: 5°C (41°F) - 40°C (104°F); SC-XX / ST-XX: 5°C (41°F) - 30°C (86°F). - The recommended relative humidity for storage is between 20% and 80%. - Check the expiry date on the container. - Misuse or incorrect application of the muscle stimulation electrodes can cause undesired effects or ineffective therapy.

Do not use electrodes that are not a suitable size or shape for your device. Always apply them correctly. - It is not recommended to use this product in patients with progressive episodes of skin irritation or contact allergies. - Stop the treatment if the patient feels discomfort or a burning sensation, and seek medical attention. Do not use the electrodes during activities that require constant attention (e.g. while operating heavy machinery, driving on a motorway, etc.). - Keep the electrodes out of the reach of children. - Follow the instructions for use and disposal of the product. - For use only on clean, unbroken skin (do not use on open wounds, lesions or infected or inflamed areas). - The product is not suitable for internal use. - In the event of accidental ingestion, consult a doctor immediately. - The electrodes are for use on one person only. They can be repositioned several times. Reuse of the electrodes for different patients can cause cross-contamination. - Do not submerge, rinse or sterilise this product, as they are procedures that can leave toxic residue or cause poor functioning. Neither the product design nor the materials with which it is manufactured are compatible with conventional cleaning and sterilisation procedures. - The temperature limits recommended for storage are: DT-XX / RT-XX / CSC-XX: 5°C (41°F) - 40°C (104°F); SC-XX / ST-XX: 5°C (41°F) - 30°C (86°F). - The recommended relative humidity for storage is between 20% and 80%. - Check the expiry date on the container. - Misuse or incorrect application of the muscle stimulation electrodes can cause undesired effects or ineffective therapy.

WASTE DISPOSAL:

Waste coming from hospitals must be disposed in accordance with regulations in force. Randomly discard is strictly forbidden. While the disposal of waste deriving residences does not require any particular precautions before product discard. It is recommended that the product is treated with disinfectant solutions (chlorine oxides, sodium hypochlorite, etc.) before being disposed.

Dispose of the packaging following the recyclable waste management protocols.

GUARANTEE - NOTICE:

TELIC, S.A.U. guarantees that the product complies with the Regulation (EU) 2017/745 on Medical devices and was manufactured

following the procedures of the TELIC, S.A.U. quality management system certified under ISO standard 13485. TELIC, S.A.U. accepts no liability for any medical costs or direct or indirect harm due to the lack of operation or incorrect operation of the products when used without observing the instructions for use. We advise you to duly report any operational problem or product fault to the Quality Assurance Department of TELIC, S.A.U. (incidents@telic.es). In the event of any serious incident related to the product, this must be reported immediately to TELIC, S.A.U. and/or the competent authority of the Member State in which the incident occurs.

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