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## EN. REUSABLE HEATING/COOLING PACK

Reusable pack for treatment with cooling or heating effect. The formulation of the gel in this product enables it to maintain the appropriate viscosity during the treatment so that the product adapts perfectly to the shape of the area to be treated.

The product is sold with the following references and formats:

| REF   | Description                                   |
|-------|-----------------------------------------------|
| FC-03 | Reusable heating/cooling pack 130 mm x 260 mm |

### INSTRUCTIONS FOR USE

#### Treatment with cooling effect

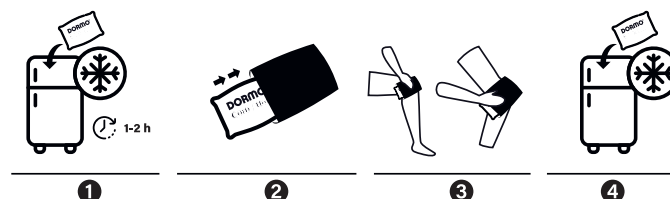
The cooling effect of the product helps reduce inflammation and pain. It is used to treat:

- Swelling
- Acute injuries (haematomas and contusions)
- Muscle or joint injuries
- Fever (it is not designed to lower the body temperature)

#### Preparación

The steps to use the reusable cooling pack are described below:

1. Place the pack in the freezer for 1 or 2 hours.
2. Remove the pack from the freezer and wrap it in the reusable cover or a dry towel.
3. Apply to the affected area.
4. The pack can be returned to the freezer for storage.

#### Treatment with heating effect

The heating effect of the product helps reduce muscle and joint pain. It is also suitable for lower back pain, torticollis, rheumatism and hypothermia. It is used to treat:

- Neck or shoulder pain
- Chills (used as a heater)
- To relieve muscle tension due to stomachache

#### Preparation

| Bain-marie                                                                                                                                                                                                                                                                                                                  | Microwave                                                                                                                                                                                                                                                                                                                                                                          |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <ol style="list-style-type: none"><li>1. Bring water to the boil in a pan.</li><li>2. Remove from the heat.</li><li>3. Submerge the pack in the hot water for 5 to 10 minutes.</li><li>4. Remove the pack from the water and wrap it in the reusable cover or a dry towel.</li><li>5. Apply to the affected area.</li></ol> | <ol style="list-style-type: none"><li>1. Submerge the bag in water completely in a microwave-appropriate container.</li><li>2. Heat at intervals of 30 seconds at maximum power (800-900 W) until the desired temperature is reached.</li><li>3. Remove the pack from the water and wrap it in the reusable cover or a dry towel.</li><li>4. Apply to the affected area.</li></ol> |

### TIME FOR USE

For short-term use. The product maintains its cooling/heating effect for 30 minutes. If you wish to prolong the treatment, repeat the preparation process. When using the cooling effect, do not apply for more than 20 minutes. Wait a further 20 minutes before using the product again in the same area.

### WARNINGS

Do not apply the hot or cold pack directly to the skin. This can cause injury. - For external use only. Do not open the pack. - Do not use on irritated or sensitive skin. - Do not place the pack directly in the microwave. - If the pack submerged in water starts to inflate, stop heating immediately. - Do not heat in the oven or grill. - Dispose of damaged packs. - Product harmful if swallowed. Do not ingest allow it to come into contact with foods. - Keep out of reach of children. - In the event of circulation problems, consult your doctor before using the pack. - If you are pregnant, consult your midwife or gynaecologist before using it. - The time for use indicated is approximate. It may depend on the method used and the ambient conditions. - Non-sterile product. Non-sterilisable product. - Keep in a cool, dry place, protected from sunlight. - Check the expiry date on the container. - Check with your doctor or pharmacist if you are not sure.

### DISPOSAL

- This product does not require special conditions for disposal. Follow the usual protocols for disposal.
- Recycle the packaging.

### WARRANTY

TELIC, S.A.U. guarantees that the product complies with the Regulation (EU) 2017/745 on Medical devices and was manufactured following the procedures of the TELIC, S.A.U. quality management system certified under ISO standard 13485. TELIC, S.A.U. accepts no liability for any medical costs or direct or indirect harm due to the lack of operation or incorrect operation of the products when used without observing the instructions for use. We advise you to duly report any operational problem or product fault to the Quality Assurance Department of TELIC, S.A.U. In the event of any serious incident related to the product, this must be reported immediately to TELIC, S.A.U. (incidents@telic.es) and/or the competent authority of the Member State in which the incident occurs.

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