TELIC S.A.U. guarantees that the product complies with the Regulation (EU) 2017/745 on Medical devices and was manufactured following the procedures of the TELIC S.A.U. quality management system certified under ISO standard 13485. TELIC S.A.U. accepts no liability for any medical costs or direct or indirect harm due to the lack of operation or incorrect operation of the products when used without observing the instructions for use. We advise you to duly report any operational problem or product fault to the Quality Assurance Department of TELIC S.A.U. in the event of any serious incident related to the product. This must be reported immediately to TELIC S.A.U. (Incidents@telic.es) and/or the competent authority of the Member State in which the incident occurs.

WARRANTY

- Dispose off as hospital contaminated waste, in accordance with regulations in force.
- Once you have removed the plastic sheath from the electrodes, dispose of it immediately in the appropriate waste container.
- Recycle package

DISPOSAL

- Do not use the electrodes if the pouch is found undamaged.
- Use electrodes with tab connection, only in combination with approved patient cables, adapters and ECG monitoring devices. Never allow contact with an external electrical source.
- In the event of use over an extended period, check for secure adhesion and absence of skin reaction on a daily basis
- Electrodes may not be repositioned
- During surgical procedures electrodes should be placed as far as possible from the electro- surgical area to minimize unwanted RF current flow and burnings in the patient.

PRECAUTIONS

- For use only on intact, clean skin (do not use on open wounds, lesions, irritated or inflamed areas).
- And use. Any allergic or other skin reactions should be immediately evaluated by a physician.
- The product is not for internal use, in case of accidental ingestion of the electrode consult a physician immediately.
- Do not use on the face, neck, or scalp. Do not soak, immerse, or sterilize the device, as these procedures may leave harmful residues or cause malfunctioning of the device. The design and materials used are not compatible with conventional cleaning and sterilization procedures.
- Gentle retrieval of electrodes from the pouch to be ensured so as to prevent any injury to the fingers as plastic cover is fine cut.
- Careless removal of electrode may cause damage to the skin, should the electrode be difficult to remove, use water to dissolve the adhesive.
- Do not use if the gel is dry or discoloured.

— ECG electrodes should only be used by or in consultation with a health care provider familiar with their proper placement

- WARNINGS**
- If they are stored in this way, the electrodes can be used safely up to 7 days after the bag has been opened.
 - If they are stored in this way, the electrodes can be used safely up to 7 days after the bag has been opened.

STORAGE

ECG Electrodes may be used on patients for up to 24 hours.

DURATION OF USE

- ① Open the pouch along the indicated line.
- ② Determine the application sites. Remove any excessive hair and ensure that the skin is clear of any oils, creams, fluids, dirt, or other impediments to good electric contact.
- ③ If necessary, clean the skin with appropriate preparation material. Always ensure the application site is dry prior to application of the electrodes.
- ④ Connect the lead wires to the electrodes and peel the plastic liner.
- ⑤ Place the electrodes on the skin. Smooth the backing surface with the fingers as necessary.
- ⑥ Turn off equipment on and perform the ECG procedure.
- ⑦ After monitoring, lift the tab to gently remove the electrode from the skin.
- ⑧ Remove gel residue, if required, with a paper towel or water.

AREA FOR APPLICATION / CONTRAINDICATIONS

INSTRUCTIONS FOR USE

- For use in passive sensing of bio-electric signals from the intact skin.
- Due to high adhesion strength of the adhesive material, this product is not suitable for use on patients with fragile skin. Not suitable for use on newborn or premature infants.

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