TRANSONIC GEL

ES. GEL PARA ULTRASONIDOS

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TRANSONIC GEL

EN. ULTRASOUND GEL

Gel for ultrasound diagnosis and therapy. Intended to be used on mucous membranes and in natural body orifices.

REF				ml	Packaging	Dispenser
Blue	G-15	Clear	GC-15	250	Bottle	
	G-15/05		GC-15/05	500	Bottle	
	G-15/1		GC-15/1	1000	Bottle	
	G-15/5		GC-15/5	5000	Flexible container	
	G-15/5RB		GC/15/5RB	5000	Rigid container	

INSTRUCTIONS FOR USE

1. Clean and dry the area of application, if necessary.
 2. Apply to the selected areas, spreading the gel evenly.
 3. Remove with a disposable towel and dispose of the product in accordance with the corresponding protocol.
- In the case of 5L format, before use, replace the cap of the container with the dosing pump or nozzle and refill the 250 ml bottle provided with the product.

TIME FOR USE

The maximum time for use is 60 minutes.

WARNINGS

This product must be used by authorised medical staff or by accredited personnel. Follow the instructions for use and disposal of the product. If possible, do not apply the ultrasound gel to damaged skin. Improper use or incorrect application of ultrasound gel could cause an incident or ineffective therapy. Single-use product. Do not re-use. Do not sterilize.

The re-use or sterilization of this product could cause:

- Cross infection due to the presence of biological waste;
- Alterations to materials;
- Loss of functionality of the product.

Sterilization of the product is not guaranteed.

Do not use if the individual packaging is open or damaged.

After having checked the integrity of the packaging, verify its content. If the packaging shows signs of damage or visible defects, do not use the product and return it to TELIC, S.A.U. Check the expiry date on the packaging. Do not use after the expiry date.

The recommended storage conditions are between 5° C and 40° C (41° F - 104° F).

DISPOSAL

- Dispose of the product following the hospital protocol for contaminated waste, in accordance with the current standards.
- Dispose of the packaging following the guidelines for the management of recyclable waste.

WARRANTY

TELIC, S.A.U. guarantees that the product complies with the Regulation (EU) 2017/745 on Medical devices and was manufactured following the procedures of the TELIC, S.A.U. quality management system certified under ISO standard 13485. TELIC, S.A.U. accepts no liability for any medical costs or direct or indirect harm due to the lack of operation or incorrect operation of the products when used without observing the instructions for use. We advise you to duly report any operational problem or product fault to the Quality Assurance Department of TELIC, S.A.U. (incidents@telic.es). In the event of any serious incident related to the product, this must be reported immediately to TELIC, S.A.U. and/or the competent authority of the Member State in which the incident occurs.

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