

DORMO[®]-SPEC

EN. DISPOSABLE SPECULUM FOR OTOSCOPY

Disposable speculum for ear examinations. Sold with the following reference and characteristics:

REF	Diameter (mm)	Patient	Company	Compatibility
4100	4,0	Adult	Welch-Allyn	All diagnostic otoscopes

INSTRUCTIONS FOR USE

Before using the product, select the appropriate size and model for the otoscope used.

1. Fit the speculum on the diagnostic otoscope.
2. After lighting up the device, insert the speculum into the patient's outer ear for the examination.
3. Upon completion, remove the speculum from the otoscope and dispose of it in accordance with the protocol established.

TIME FOR USE

For transient use during the ear examination.

WARNINGS












This product must be used by medical staff. – Follow the instructions for use and disposal of the product. – Verify compatibility with the device. – Single-use product, to be used only for the time needed for the ear examination. The reuse of this product can cause cross contamination. – Non-sterile product. Non-sterilisable product. – Keep the product in its original dispenser case. – Check the expiry date on the container.

DISPOSAL

- Dispose of the product following the hospital protocol for contaminated waste, in accordance with the current standards.
- Dispose of the packaging following the recyclable waste management protocols.

WARRANTY

TELIC, S.A.U. guarantees that the product complies with the Regulation (EU) 2017/745 on Medical devices and was manufactured following the procedures of the TELIC, S.A.U. quality management system certified under ISO standard 13485. TELIC, S.A.U. accepts no liability for any medical costs or direct or indirect harm due to the lack of operation or incorrect operation of the products when used without observing the instructions for use. We advise you to duly report any operational problem or product fault to the Quality Assurance Department of TELIC, S.A.U. In the event of any serious incident related to the product, this must be reported immediately to TELIC, S.A.U. (incidents@telic.es) and/or the competent authority of the Member State in which the incident occurs.

	CE marking		Keep dry
	Latex free		Adult
	Consult instructions for use		Paediatric
	Medical device		Importer
	Do not re-use		Manufacturer
	Keep away from sunlight		

Sponsored by:



Stark Medical Pty Ltd - ABN: 14619334771
Unit 5, 6-8 McLachlan Ave
Artarmon, NSW 2064 Australia
Ph: 1300 938 637
contact@starkmed.com.au
starkmed.com.au



TELIC, SAU
C/ Molí d'en Barri, 7
08415 Bigues i Riells
Barcelona, Spain
www.telic.group

WITH THE GUARANTEE OF
TELIC GROUP