ELECTRO-GEL

ES. GEL CONDUCTOR PARA ELECTRODOS

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ELECTRO-GEL

EN. CONDUCTIVE GEL FOR ELECTRODES

Conductive gel for the passive detection of bioelectric signals.

The product is sold in the following formats and references

REF	Description
G-10	250 ml bottle with dispensing cap

INSTRUCTIONS FOR USE

1. Clean and dry the area of application, if necessary.
2. Apply a small amount of gel to the conductive area of the electrode.
3. Perform the monitoring procedure in accordance with the usual protocols.
4. Detach the electrodes, remove the remaining gel with a disposable towel and dispose of the product in accordance with the corresponding protocol. If the electrodes are reusable, follow the maintenance instructions indicated by the manufacturer.

TIME FOR USE

In accordance with the hospital procedures, the ECG electrodes must be replaced if a deterioration in monitoring quality is detected after a maximum of 24h.

WARNINGS

The ECG gel must only be used by a health professional familiar with its proper use and application, or under their direction.- Do not use on electrodes with preapplied gel. - If an allergy or other skin reaction occurs, the patient must be assessed by a doctor immediately. - Follow the instructions for use and disposal of the product. - For use on clean, unbroken skin only (do not use on open wounds, lesions or infected or inflamed areas). - The product is for external use only.- In the event of accidental ingestion, consult a doctor immediately. - Single-use gel. Reuse for different patients can cause cross-contamination. - Do not submerge, rinse or sterilise this product, as they are procedures that can leave toxic residue or cause poor functioning. Neither the product design nor the materials with which it is manufactured are compatible with conventional cleaning and sterilisation procedures. - Do not use the product if the gel is dry or has changed colour. - The recommended temperature limits for storage are between 5°C (41°F) and 40°C (104°F). - Check the expiry date on the container. - Misuse or improper application of the ECG gel can cause an incident or ineffective therapy.

DISPOSAL

Dispose of the product following the hospital protocol for contaminated waste, in accordance with the current standards.

Dispose of the packaging following the recyclable waste management protocols.

WARRANTY

TELIC, S.A.U. guarantees that the product complies with the Regulation (EU) 2017/745 on Medical devices and was manufactured following the procedures of the TELIC, S.A.U. quality management system certified under ISO standard 13485. TELIC, S.A.U. accepts no liability for any medical costs or direct or indirect harm due to the lack of operation or incorrect operation of the products when used without observing the instructions for use. We advise you to duly report any operational problem or product fault to the Quality Assurance Department of TELIC, S.A.U. (incidents@telic.es). In the event of any serious incident related to the product, this must be reported immediately to TELIC, S.A.U. and/or the competent authority of the Member State in which the incident occurs.

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