ELECTRO-GEL

ES. GEL CONDUCTOR PARA ELECTRODOS

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ELECTRO-GEL

EN. CONDUCTIVE GEL FOR ELECTRODES

Conductive gel for electro-medical applications (ECG, TENS).

The product is marketed with the format and reference below.

REF	Description
G-10	250 ml bottle with dispensing cap

INSTRUCTIONS FOR USE

1. Clean and dry the area where the gel is to be applied, if applicable.
2. Apply a small amount of gel to the conductive surface of the electrode.
3. Connect the electrodes to the device to carry out the procedure as per the usual protocols.
4. Remove the electrodes, wipe off the gel with a single-use cloth then dispose of the product in accordance with the corresponding protocol. If the electrodes can be reused, follow the cleaning instructions given by the manufacturer.

DURATION OF USE

The electrodes should be replaced if a decrease in quality is detected or at the latest after 24 hours.

WARNINGS

The gel should only be used by or under the supervision of a healthcare professional who knows how to apply and properly use it. – Do not use on pre-gelled electrodes. – If an allergy or any other skin reaction occurs, it must be immediately evaluated by a doctor. – Follow the instructions for use and the instructions for disposal of the product. – For use on healthy and clean skin only (do not use on wounds, injuries, infected areas or skin inflammations). – The product is not intended for internal use. – Consult a doctor immediately in case of accidental ingestion. – Single-use gel. Reuse on different patients can result in cross infections. – Do not immerse, rinse or sterilise this product, as these procedures can leave harmful residues or cause a dysfunction. Neither the design of the product nor the materials with which it is manufactured are compatible with conventional cleaning and sterilisation procedures. – Do not use the product if the gel is dry or has changed colour. – The recommended temperature limits for its storage are between 5°C (41°F) and 40°C (104°F) – Check the expiry date printed on the packaging. – Improper use or incorrect application of the gel can result in incidents and/or make the treatment ineffective.

DISPOSAL

The product must be disposed of in accordance with current standards. Dispose of the packaging as per the regulations on the management of recyclable waste.

GUARANTEE INFORMATION

TELIC, S.A.U. guarantees that the product complies with regulation (EU) 2017/745 on medical devices and that it was manufactured in accordance with the procedures of the TELIC, S.A.U. quality management system certified as per standard ISO 13485. TELIC, S.A.U. declines any responsibility for medical costs and direct or indirect damages resulting from a dysfunction or incorrect functioning of the products when they are used without complying with the instructions for use. We recommend reporting any dysfunction or defect of the product to the TELIC, S.A.U. quality assurance department as quickly as possible (incidentes@telic.es). Any serious incident related to the product should also be reported immediately to TELIC, S.A.U. and/or to the competent authority of the Member State in which the incident occurred.

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